



# Sporicide Plus™



## Hospital Grade Disinfectant Sporicidal Disinfectant Cleaner

A powerful, one step hospital grade disinfectant cleaner that is effective against *Clostridium difficile* spores utilising Accelerated Hydrogen Peroxide Technology (AHP®).

### Features

- Ready- to- use Sporicidal Surface Disinfectant for hard non-porous surfaces
- Effective against:
  - **Spores - *Clostridium difficile***, *Clostridium sporogenes* and *Bacillus subtilis*
  - **Virus - Poliovirus**
  - **Antibiotic Resistant Bacteria - MRSA, VRE**
  - **Bacteria - Staphylococcus aureus, Proteus vulgaris, Pseudomonas aeruginosa, Salmonella choleraesuis, Enterococcus hirae and E. coli**
  - **Fungi - Trichophyton mentagrophytes**
- Utilises Accelerated Hydrogen Peroxide Technology (AHP®)

### Benefits

- Delivers fast, effective performance in one step
- Effective against a broad spectrum of pathogens
- Post use the active ingredient breaks down to environmentally friendly oxygen and water
- Easy to use – apply, wipe and rinse

### Designed for

- Healthcare
- Medical Establishments



AUST R 284935





# Sporicide Plus™

## Hospital Grade Disinfectant Sporicidal Disinfectant Cleaner

### Use instructions

Appropriate Personal Protection must be worn during all cleaning and disinfection procedures. Remove any gross soil. Apply Sporicide Plus undiluted to surface. Allow 1 minute contact time for *C. difficile* disinfection.

### Other organisms:

Disinfectant Times	Bactericidal	Virucidal	Fungicidal
No gross soil	1 minute	1 minute	1 minute
In the presence of gross soil	5 minutes (MRSA, VRE) 10 minutes (others)	1 minute	1 minute

Always check surface compatibility. Do not use on glassware, dishes or utensils. Not recommended for use on brass, copper or marble. It is advisable to wipe surfaces with damp cloth after disinfection and allow to air dry. Not intended to be used on therapeutic devices. Do not mix with detergents or other chemicals. Store below 25°C.

### To remove Sporicide Plus Wipes from canister:

1) Remove lid 2) Remove the seal 3) Using a glove pull up corner of first wipe from centre of roll and push 1 to 2 cm through "X" in lid 4) Replace lid 5) Lift lid top and pull up wipes one at a time. To close between uses to prevent drying, simply keep lid flip top down and ensure wipe corner is inside the raised circle in the lid.

### Technical data

Appearance	Clear, colourless	Specific gravity (20°C)	1.0325
Odour	Characteristic	pH	2.7

*The above data is typical of normal production and should not be taken as a specification.*

Product	Pack size	SKU code	Availability
Sporicide Plus Wipes*	12 x 160 wipes	100973031	ANZ – Made to Order

### Safe handling and storage information

Full guidance on the handling and disposal of this product is provided in a separate Safety Data Sheet. Store in original container. Avoid extremes of temperature and humidity. Store below 25°C.

### Safety Reminder

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use, and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDSs are available online at [www.diversey.com](http://www.diversey.com) or by calling Toll free: Australia, 1800 647 779 or New Zealand, 0800 803 615.

For additional information, please contact us at:  
Diversey Australia Pty Ltd Toll free: 1800 647 779, Diversey New Zealand Ltd Toll free: 0800 803 615 [www.diversey.com](http://www.diversey.com).

Legal disclaimer: The associated products or services are manufactured by or otherwise emanate from Diversey Companies and their subsidiaries and not from Sealed Air.

© 2019 Diversey, Inc. All Rights Reserved.