

# Oxivir® Five 16



## Hospital Grade One-Step Cleaner Disinfectant

	<p>1. Fill bottle with Oxivir® Five 16</p> <p><b>Dilution:</b></p> <table border="0"> <tr> <td><b>3.78L</b></td> <td></td> <td><b>J-Fill™</b></td> </tr> <tr> <td>1:16</td> <td>63ml/L</td> <td>1:16</td> </tr> <tr> <td>1:64</td> <td>16ml/L</td> <td></td> </tr> <tr> <td>1:128</td> <td>8ml/L</td> <td></td> </tr> <tr> <td>1:256</td> <td>4ml/L</td> <td></td> </tr> </table>	<b>3.78L</b>		<b>J-Fill™</b>	1:16	63ml/L	1:16	1:64	16ml/L		1:128	8ml/L		1:256	4ml/L		
<b>3.78L</b>		<b>J-Fill™</b>															
1:16	63ml/L	1:16															
1:64	16ml/L																
1:128	8ml/L																
1:256	4ml/L																
	<p>2. Apply use solution onto surface</p>																
	<p>3. All surfaces must remain visibly wet for 10 minutes</p>	<p><b>AUST L 202507</b></p>															
	<p>4. Wipe surfaces and let air dry</p>	<p><b>Use for cleaning and disinfecting:</b></p> <p>Hard Surfaces      Floors</p>															
<p><b>For best result:</b></p> <ul style="list-style-type: none"> <li>Use with TASKI® Jonmaster microfibre cloths/mops</li> </ul>	<p><b>Effective</b> against 19 bacteria including <b>VRE, MRSA, MRSE, PRSP, Escherichia coli, Acinetobacter baumannii, Klebsiella pneumoniae, Listeria monocytogenes, Salmonella choleraesuis</b></p>	<p><b>Keeping Safe:</b></p> <ul style="list-style-type: none"> <li>Wear gloves for prolonged use</li> <li>Do not mix with other detergents or chemicals</li> <li>Discard diluted solution after use</li> </ul>															