




Oxivir® Tb Wipes



Hospital Grade One-Step Cleaner Disinfectant

	<ol style="list-style-type: none"> 1. Pull towelette from the canister dispenser 2. Wipe the surface <p>Oxivir® Tb will not damage fabric or irritate skin</p> <p>Oxivir® Tb breaks down to oxygen & water</p>	<p>TGAEPA Reg. 70627-60</p> <p>Tested & proven effective against a wide spectrum of pathogens:</p> <p>Viruses:</p> <ul style="list-style-type: none"> • Hepatitis B • Hepatitis C • Herpes Simplex Virus, Type 1 & 2 • HIV-1 (AIDS virus) • Human Coronavirus • Influenza A • Norovirus • Poliovirus • Rotavirus • Adenovirus • Rhinovirus Type 14 • H1N1 Influenza A (Swine flu) • Avian influenza A • Feline Calicivirus <p>Bacteria:</p> <ul style="list-style-type: none"> • Pseudomonas aeruginosa • Salmonella enterica • Staphylococcus aureus • Acinetobacter baumannii • Escherichia coli O157:H7 • Klebsiella pneumoniae • Escherichia coli with extended beta-lactamase resistance (ESBL) • Shigella Dysenteriae <p>Antibiotic- Resistant Bacteria:</p> <ul style="list-style-type: none"> • Enterococcus faecalis (VRE) • Staphylococcus aureus (MRSA) • Staphylococcus aureus (CA-MRSA) • Staphylococcus aureus (CA-MRSA) <p>Fungi: Trichophyton mentagrophytes</p> <p>Tb: Mycobacterium bovis</p>
	<ol style="list-style-type: none"> 1. All surfaces must remain visibly wet for 1 minute for viruses and bacteria. Allow 5 minutes for Tb and 10 minutes for Fungi. 2. Allow to air dry 3. Dispose wipe after use. 4. Do not flush in a toilet 	
	<ol style="list-style-type: none"> 1. Ideal for keyboards, telephones, patient areas and high-touch surfaces 2. If disinfecting food contact surfaces, rinse with water. 	
<p><u>SUPPLIES/TOOLS:</u></p> <ul style="list-style-type: none"> • Gloves should be considered, as prolonged hand contact will dry skin 	<p><u>SAFETY:</u></p> <p>No PPE Gear Needed.</p> <ul style="list-style-type: none"> • Oxivir Tb has a low concentration of active (AHP) • There is no normal need to use Personal Protective Equipment (e.g. safety eyewear) 	<p><u>CONTROL POINTS:</u></p> <ul style="list-style-type: none"> • Do not mix with other detergents or other chemicals • Not for use on glasses, dishes and utensils or therapeutic devices. • Not recommended for use on brass, copper or marble • Safe on carpets, fabrics & clothes • Safe on stainless steel • Biodegrades to OXYGEN and WATER