

# The VeriClean™ System

## Marking High Touch Surfaces\* with the VeriClean™ Fluorescent Spray:

Hold the VeriClean Fluorescent Marking Spray 15-23cm away from the surface to be marked.  
 Press the pump once to dispense a small amount of liquid marker on the surface.  
 Allow the surface to dry.  
 Using the **Patient Room Audit Form**, check "Removed" to capture the marked surfaces or  
 Using the **iMAP Audit Tool**, select "Yes" to capture the marked surfaces.



## Patient Care Surfaces:

Call button/remote control



Telephone



Bed rail & bed control panel



Patient over bed table



Bedside table



Patient / visitor chair



## Restroom Surfaces:

Toilet flush handle



Toilet hand holds / rails



Patient pull cord



Wall area around toilet



Toilet seat, bowl and booster




Wall area around toilet



Restroom door handle



Sink and faucet handles

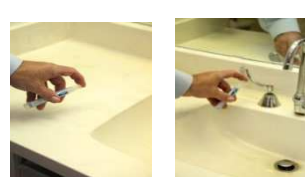

Portable commode



Restroom light switch



Room door handle/knob



Room light switch



\*Note: Your facility may have additional surfaces to be tested

## Room Entrance Surfaces: