



Sporicide Plus™ Wipes



Disinfectant Wipe - Hospital Grade Ready to use Sporicidal Disinfectant Cleaner

A powerful, One Step Hospital Grade Disinfectant Cleaner that is effective against *Clostridium difficile* spores utilising Accelerated Hydrogen Peroxide Technology (AHP®) Technology.

Features

- Ready- to- use Sporicidal surface cleaner and disinfectant for hard non-porous surfaces
- Effective against:
 - **Spores** - *Clostridium difficile*
 - **Virus** - Poliovirus.
 - **Antibiotic Resistant Bacteria** – VRE
 - **Bacteria** - Staphylococcus aureus, Pseudomonas aeruginosa, S. choleraesuis and E. coli
 - **Fungi** - Trichophyton mentagrophytes

Benefits

- Delivers fast, effective performance in one step
- Sporicidal in *1 minute
- Easy to use – apply, wipe and allow to air dry
- Effective against a broad spectrum of pathogens
- Post use the active ingredient breaks down to environmentally friendly oxygen and water

Designed for

- Healthcare
- Medical Establishments



AUST L 284935



Sporicide Plus™ Wipes



Efficacy Data:

Sporicide Plus Wipes are proven to kill spore-forming bacteria, gram-positive and gram-negative bacteria, enveloped and non-enveloped viruses and fungi.

Organism		Contact Time
Spores	• Spores of Clostridium difficile	• 1 min*
Bacteria	Gram-positive	• Vancomycin-resistant enterococci (VRE)
		• Staphylococcus aureus
		• Enterococcus hirae
	Gram-negative	• Escherichia coli
		• Pseudomonas aeruginosa
		• Salmonella choleraesuis
Virus	• Poliovirus type 1	• 1 min
Fungi	• Trichophyton mentagrophytes	• 1 min

* No gross soil

Directions for Use:

How to Use Sporicide Plus™ Wipes

Put on PPE.

Remove one wipe from canister. Close Lid.

Work from clean to dirty in an S pattern. Allow the surface to air dry.

Dispose used wipes into a suitable waste bin.

READY TO USE PRE-WETTED DISINFECTING SPORICIDAL WIPE

Appropriate Personal Protection Equipment (PPE) must be worn during all cleaning and disinfection procedures. *Remove any gross soil. Apply Sporicide Plus Wipes undiluted to surface. Allow 1 minute contact time.

Do not use on glassware, utensils or dishes. Not recommended for use on brass, copper or marble. Hard surface disinfectant only. Not to be used on skin. Not intended to be used on medical devices or other therapeutic goods. Store below 25°C.



Sporicide Plus™ Wipes



To remove Sporicide Plus Wipes from canister:

- 1) Remove lid
- 2) Remove the seal
- 3) Using a glove, pull up corner of first wipe from centre of roll and push 1 to 2 cm through "X" in lid
- 4) Replace lid.
- 5) Lift lid top and pull up wipes one at a time. Close between uses to prevent drying, simply keep lid flip top down and ensure wipe corner is inside the raised circle in the lid.

Note: This product is compatible with a wide range of materials commonly found in healthcare however may not be suitable for all. If you are uncertain about the material composition of an item, confirm with the manufacturer prior to product use or rinse to remove product residue. Prolonged exposure to copper, brass, and anodized aluminium should be avoided. Certain grades of plastic may need to be rinsed after use to prevent streaking.

Technical data

Appearance	Clear, colourless	Specific gravity (20°C)	1.03
Odour	Characteristic	pH	2.7 (neat liquid) approx.

The above data is typical of normal production and should not be taken as a specification.

Product	Pack size	SKU code	Availability
Sporicide Plus Wipes	12 x 160 wipes	100973031	AU, NZ

Accelerated Hydrogen Peroxide® and design are registered trade marks of Virox Technologies Inc., used under license. All rights reserved.

Safe handling and storage information

Full guidance on the handling and disposal of this product is provided in a separate Safety Data Sheet. Store in original container. Avoid extremes of temperature and humidity. Store below 25°C.

Safety Reminder

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use, and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDSs are available online at www.diversey.com or by calling Toll free: Australia, 1800 647 779 or New Zealand, 0800 803 615.

For additional information, please contact us at:
Diversey Australia Pty Ltd Toll free: 1800 647 779, Diversey New Zealand Ltd Toll free: 0800 803 615 www.diversey.com.

Legal disclaimer: The associated products or services are manufactured by or otherwise emanate from Diversey Companies and their subsidiaries and not from Sealed Air.

© 2019 Diversey, Inc. All Rights Reserved.